

# CELOX Trauma Gauze Instructions for Use

**INSTRUCTIONS FOR USE:**  
For moderate to severe bleeding

# 1 – Tear open CELOX Trauma Gauze pack



## 2 – Unravel CELOX Trauma Gauze



## 3 – Pack unraveled CELOX Trauma Gauze into wound



## 4 – Apply FIRM pressure directly to wound for 3 minutes



## 5 – If bleeding persists

**If any bleeding persists, apply direct pressure for an additional 3 minutes**

**An additional CELOX Trauma Gauze may be used**

6 – Wrap and tie bandage so as to maintain pressure on the wound



## 7 – Once bleeding is controlled

**Discard any gauze that has not been used to pack the wound**

**Seek MEDICAL CARE as soon as possible**

**Show empty pack to medical personnel**

**INSTRUCTIONS FOR USE:**  
For pre-hospital management of  
1st and 2nd degree burns

# 1 – Irrigate wound area according to locally established protocols



## 2 – Tear open CELOX Trauma Gauze pack



### 3 – Unravel CELOX Trauma Gauze and cover burn, overlapping at least 2 inches onto the surrounding skin



## 4 – Apply water or saline solution to CELOX Trauma Gauze



## 5 – Cover CELOX Trauma Gauze with a sterile, loosely woven gauze pad and secure



## 6 – Once burn dressing is applied

**Discard any gauze that has not been used on wound**

**Seek MEDICAL CARE as soon as possible**

**Show empty pack to medical personnel**

# CELOX GAUZE

**CELOX controls life threatening bleeding fast**



**CELOX Trauma Gauze is a dual purpose hemostatic gauze used to both effectively control major bleeding and to help protect 1<sup>st</sup> and 2<sup>nd</sup> degree burns**

## **Available Sizes**

One Gauze Strip:  
3IN x 6FT (7.6CM x 1.8M)

## **Specs**

Package Dimensions:  
4IN x 7IN (10.2CM x 17.8CM)

## **Shelf Life:**

3 years

# Questions?

For more information, please visit:

**[www.sammedical.com/celox](http://www.sammedical.com/celox)**