

CELOX-A Instructions for Use

1 – Obtain CELOX-A packet



2 – Tear open CELOX-A packet



3 – Select applicator



4 – Remove clear plastic tab from applicator



5 – Insert plunger into barrel of applicator



6 – Remove blue plastic cap from end of applicator, just prior to use



7 – Insert applicator with attached plunger as far into wound as possible



8 – Slowly push plunger in as you withdraw applicator from the wound



9 – Cover wound with gauze and apply FIRM pressure for 5 minutes



10 – If bleeding persists

If any bleeding persists:

- **use a second applicator**
- **apply pressure for an additional 5 minutes**

11 – Wrap and tie bandage so as to maintain pressure on the wound



12 – Once bleeding is controlled


Seek MEDICAL CARE as soon as possible

Show empty pack to medical personnel

Discard any unused portion or partially used applicators

CELOX-A

CELOX controls life threatening bleeding fast

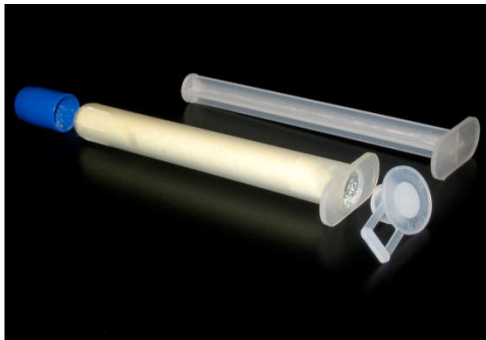

The CELOX-A applicator and plunger delivery system makes it possible for the hemostatic agent to reach deep into small penetrating wounds, getting straight to the source of bleeding

Available Sizes

6g (0.2oz) Applicator/Plunger Set

Specs

Package Dimensions:
4" x 8.875"

Shelf Life:

3 years

More questions?

For more information, please visit:

www.sammedical.com/celox