

# CELOX GAUZE Instructions for Use

# 1 - Obtain CELOX GAUZE package



## 2 - Tear open CELOX GAUZE pack



# 3 - Unravel CELOX GAUZE



## 4 - Pack unraveled CELOX GAUZE into wound



5 - Apply FIRM pressure directly to wound for 5 minutes



## 6 - If bleeding persists

**If any bleeding persists, apply direct pressure for an additional 5 minutes**

**An additional CELOX GAUZE may be used**

7 - Wrap and tie bandage so as to maintain pressure on the wound



## 8 - Once bleeding is controlled

**Discard any gauze that has not been used to pack the wound**

**Seek MEDICAL CARE as soon as possible**

**Show empty pack to medical personnel**

# CELOX GAUZE

**CELOX controls life threatening bleeding fast**



**CELOX GAUZE** combines our ground breaking CELOX agent with a fabric dressing, allowing for easy and precise placement

## Available Sizes

One Gauze Strip:  
3IN x 10FT (7.6CM x 3.0M)

## Specs

Package Dimensions:  
7IN x 4.3IN

## Shelf Life:

3 years

# More questions?

For more information, please visit:

**[www.sammedical.com/celox](http://www.sammedical.com/celox)**