

CELOX Instructions for Use

1 - Obtain CELOX packet



2 - Tear open CELOX packet



3 - Blot away excess blood with gauze pad



4 - Pour entire contents of CELOX pouch directly into wound



5 - Place unraveled gauze on top of wound



6 - Apply FIRM pressure directly to wound for 5 minutes



7 - If bleeding persists

If any bleeding persists, apply direct pressure for an additional 5 minutes

8 - Wrap and tie bandage so as to maintain pressure on the wound



9 - Once bleeding is controlled

Discard any unused portion

Seek MEDICAL CARE as soon as possible

Show empty pack to medical personnel

CELOX

CELOX controls life threatening bleeding fast



CELOX is the original comprised of absorbent hemostatic granules that can be poured directly into a wound. This presentation provides the most diverse set of uses

Available Sizes

35g Pouch
15g Pouch

Specs

Package Dimensions:
35g: 5" x 7.5"
15g: 3.75" x 5.5"

Shelf Life:

3 years

More questions?

For more information, please visit:

www.sammedical.com/celox